









Document ref IFU-024 rev 0 01/06/2022	INSTRUCTIONS FOR USE FOR THE CARE, HANDLING AND REPROCESSING OF RE-USABLE CONTAINERS / HOLLOWARE			
DEVICE(S)	• These instructions apply to re-usable containers and holloware supplied by Bolton Surgical Limited including: Electrosurgery quivers, general purpose bowls, sterilisation/disinfection containers, sieves etc.		INSTRUMENT CLASSIFICATION	Class 1
INTENDED USE	• Portable receptacles/containers of various forms intended for a variety of medical purposes including containing fluids, straining fluids (eg. urine), collecting body waste or other matter, carrying or holding instruments prior to or during a procedure, laying out sterile surgical instruments, sutures and other utensils/items during an operation or intervention or, holding wrapped and/or unwrapped medical/dental devices in a chemical washer/disinfector and/or sterilizer during the washing/disinfection and/or sterilization cycle(s). The container may also be used for transport and storage of the processed devices.			
HOW SUPPLIED	 Bolton Surgical re-usable containers and holloware are LATEX FREE , including their packaging.	 Bolton Surgical re-usable containers and holloware are supplied non-sterile and must be cleaned and sterilised prior to each use.		
 WARNINGS AND CONTRAINDICATIONS	<ul style="list-style-type: none">• WARNING: If this device is/was used on a patient with, or suspected of having Creutzfeld Jakob Disease (CJD), the device cannot be re-used and must be destroyed due to the inability to reprocess or sterilise to eliminate cross-contamination risk.• These items are intended for use only by appropriately qualified surgical practitioners.• Do not use the device if, in the opinion of the surgical practitioner, patient safety risks outweigh the benefits.• Risk of infection – Do not use any surgical device showing signs of corrosion or inadequate decontamination.• Do not use damaged Instruments.• Follow instructions and warnings as issued by manufacturers of any decontaminants, disinfectants and cleaning agents used. Wherever possible avoid use of mineral acids and harsh, abrasive agents.• No part of the process to exceed 140°C.• When handling biologically contaminated instruments and reprocessing medical devices always handle with care wearing appropriate protective gloves, eyewear and clothing in accordance with local Health & Safety procedures.			
LIMITATIONS OF USE AND REPROCESSING	<ul style="list-style-type: none">• Repeated reprocessing has minimal effect on the service life of containers and holloware. End of useful service life is normally determined by wear and damage in use. (See 'INSPECTION' below)• Misuse can result in over-stressing the device causing distortion or cracks or other irreparable damage.			
INSTRUCTIONS:				
PREPARATION FOR FIRST USE	<ul style="list-style-type: none">• Before first use, the re-usable device(s) covered by this IFU must be cleaned, inspected and sterilised in accordance with the Instructions below.			
FROM POINT OF USE	<ul style="list-style-type: none">• At point of use, remove gross soil using absorbent wipes. Wherever possible, do not allow blood, surgical debris or bodily fluids to dry on the instruments.• For best results and to maximise their service life reprocess immediately after use to minimise the potential for drying before cleaning.• If transfer to reprocessing is likely to take time, consider covering the instruments with a damp cloth or use an enzymatic foam spray cleaner to help prevent soil from drying or, soak in an enzymatic solution (prepared according to the manufacturer's instructions) to help facilitate cleaning.• Do not leave instruments soaking in saline or chlorinated solutions.• Avoid mechanical damage during transportation to the processing area (e.g. do not mix heavy devices with delicate items). Pay particular attention whenever cutting edges are present to avoid injury and damage to or by the instrument. Separate sharp and delicate surgical instruments.• Avoid unnecessary contamination or cross contamination risk by transporting used instruments for reprocessing in closed or covered containers			
AUTOMATED CLEANING	<ul style="list-style-type: none">• Whenever possible automated cleaning methods are preferable to manual methods to provide a more consistent and reliable process and, reduce staff exposure to contaminated devices and the cleaning agents used.• Use suitably authorised washer-disinfector machines and low foaming, non-ionising cleaning agents and detergents following the manufacturer's instructions for use, warnings, concentrations and recommended cycles.• Containers and holloware covered by these instructions can withstand alkaline cleaning agents up to 12.5 pH.• Containers and holloware covered by these instructions can withstand thermal disinfection at 90°C to 95°C for a minimum of 1 minute.• Avoid contact between devices if movement during washing could cause damage or impair the washing action.• Soft, high purity water which is controlled for bacterial endotoxins or mains supplied potable tap water is suitable for use in the final rinse stage.• On completion of the cleaning cycle, visually inspect each device for dryness and any remaining soil. If soil remains, repeat the cleaning process.• Remaining wetness may be removed with medical grade compressed air or by heating in an oven below 110°C <p>Note: These instructions have been validated for the products detailed above using a washer-disinfector operated in accordance with the recommendations included in this IFU. The detergent used was 12.6pH.</p>			
MANUAL CLEANING	<ul style="list-style-type: none">• Manual cleaning is not advised if an automatic washer-disinfector is available. If this equipment is not available, use the following process:1. Use a double sink system (wash/rinse) dedicated for instrument cleaning (not used for hand washing). Ensure that the water temperature does not exceed 35°C.2. In the first sink, keeping the device submerged, with an autoclavable brush, apply suitably approved cleaning solution to all surfaces until all soil has been removed. Always brush away from the body and avoid splashing.3. In the second sink, rinse with soft, high purity water which is controlled for bacterial endotoxins or, mains supplied potable tap water so that water reaches all parts of the device, then carefully hand dry or use a drying cabinet			
CLEANING INSPECTION	<ul style="list-style-type: none">• After cleaning, visually inspect <i>all</i> surfaces. If ANY soil or fluid is still visible, return the instrument for repeat decontamination.			
MAINTENANCE	<ul style="list-style-type: none">• Proper cleaning and careful handling will ensure that containers and holloware perform as intended and will maximise their useful life.			

INSPECTION	<ul style="list-style-type: none">Visually inspect and check all container / holloware surfaces for completeness, damage, excessive wear, staining and corrosion.Remove for repair or replacement any worn out, cracked, fractured or otherwise damaged items. <p>Note: If a used instrument is returned to the manufacturer / supplier for any reason, the instrument must be decontaminated and sterilised and be accompanied with the relevant documented evidence.</p>														
PACKING FOR STERILISATION	<ul style="list-style-type: none">All instruments to be packed following local protocol or in accordance with ISO11607-1														
STERILISATION	<ul style="list-style-type: none">Use suitably authorised vacuum autoclave operating at 134°C to 137°C, with a minimum holding time of 3 minutes (see 'Additional Information' below for alternative sterilisation parameters)Always follow the instructions of the machine manufacturer.Ensure instruments are dry before sterilisation.Sterilisation cases should be loaded just prior to the sterilisation step.When sterilising multiple instruments in one autoclave cycle, ensure that the steriliser manufacturer's stated maximum load is not exceeded.														
STORAGE	<ul style="list-style-type: none">Ensure instruments are dry before storage and stored in dry, clean conditions at an ambient room temperature.														
GENERAL CLEANING PRECAUTIONS	<ul style="list-style-type: none">Do not soak surgical devices in hot water, alcohol, disinfectants or antiseptics to avoid coagulation of mucus, blood or other body fluids. Do not exceed two hours soaking in any solution.Do not use steel wool, wire brushes, pipe cleaners or abrasive detergents.The quality of the water used for diluting cleaning agents and for rinsing medical devices should be carefully considered. Mineral residues from hard water can result in staining of the device or prevent effective cleaning and decontamination. De-scaling agents, if used, will not harm the devices.If practicable, avoid processing instruments of different metallic composition close together to minimise risk of electrolytic action between the metals that can result in corrosion.														
ADDITIONAL INFORMATION	<ul style="list-style-type: none">Other forms of cleaning** (i.e. Ultrasonic) and sterilisation (i.e. Low temperature steam and Formaldehyde, Ethylene Oxide and Gas Plasma) are available. However, always follow the instructions for use as issued by the equipment manufacturer and always consult with them if in any doubt over the suitability of any process used.Alternative sterilising parameters** - vacuum autoclave operating at 132°C to 135°C, with a minimum holding time of 4 minutes.Cleaning and sterilising guidelines are available in UK Health Technical Memorandum - HTM01-01 and ISO17665-1 <p>** Products covered by this IFU have not been validated for these forms of cleaning and sterilisation.</p> <p>Note: It is the responsibility of the reprocessor to ensure that the reprocessing that is actually carried out, using the equipment, materials and personnel in the reprocessing facility, achieves the desired results. This requires validation and routine monitoring of the process. Likewise, any deviation by the reprocessor from the instructions provided must be properly evaluated for effectiveness and potential adverse consequences.</p>														
MAINTENANCE AND REPAIR	<ul style="list-style-type: none">Instruments can be returned to Bolton Surgical for repair but must be decontaminated and sterilised and be accompanied with the relevant documented evidence. Failure to supply decontamination/sterilisation certification will result in products being returned untouched for re-processing and delayed repairs.Repairs carried out by Bolton Surgical are guaranteed for 12 months to be free from defects in workmanship, materials and parts used to carry out the repair providing the instrument is used normally for its intended surgical purpose. Any repair parts or workmanship proving to be defective will be replaced or repaired, at our discretion, at no charge to the customer.														
LIMITED WARRANTY	<ul style="list-style-type: none">Bolton Surgical re-usable containers and holloware are guaranteed for a period of 15 years from the date of purchase (exclusions and terms & conditions apply) against product failure resulting from defective materials and workmanship, when used by persons with the required specialist knowledge and training, for the purpose for which the device is intended and, properly maintained in accordance with this IFU.Liability is refused for products which have been modified as compared to the originally supplied product, misused, incorrectly handled or, used for a purpose that differs in any way from product's stated Intended Use.														
RETURNED GOODS POLICY	<ul style="list-style-type: none">Customers wishing to return goods for any reason must do so in accordance with Bolton Surgical's Returns Policy (ref. POL 009) a copy of which is supplied with each order or, is available online by visiting www.boltons.co.uk.Determination of a product defect will be made by Bolton Surgical Ltd.														
DISPOSAL	<ul style="list-style-type: none">End of service life surgical instruments must be decontaminated and sterilised prior to disposal. Disposal should be in accordance with local waste management protocols.														
INCIDENT REPORTING	<ul style="list-style-type: none">Report any serious incident that has occurred in relation to the use of a surgical instrument to the manufacturer and the competent authority of the country in which the user and/or patient is established and in accordance with the reporting rules applicable in that country.														
VALIDATION	<ul style="list-style-type: none">These instructions have been validated for the products detailed above using a washer-disinfector operated in accordance with the recommendations included in this IFU. The detergent used was 12.6pH.														
EXPLANATION OF SYMBOLS USED ON LABELS	<table><tr><td></td><td></td><td></td><td></td><td></td><td></td><td></td></tr><tr><td>Manufacturer</td><td>Manufacturer's Product Code</td><td>Manufacturer's Batch Code</td><td>Supplied Non- Sterile</td><td>Supplied Latex Free</td><td>Caution</td><td>Consult Instructions for Use</td></tr></table>								Manufacturer	Manufacturer's Product Code	Manufacturer's Batch Code	Supplied Non- Sterile	Supplied Latex Free	Caution	Consult Instructions for Use
															
Manufacturer	Manufacturer's Product Code	Manufacturer's Batch Code	Supplied Non- Sterile	Supplied Latex Free	Caution	Consult Instructions for Use									
 Bolton Surgical Limited Churchill House, 16 Churchill Way, Chapeltown, Sheffield, S35 2PY, UK T: +44 (0) 114 240 44 F: +44 (0) 114 257 6555 E: sales@boltons.co.uk W: www.boltons.co.uk	   Manufactured under an ISO 13485 registered Quality Management System														
2023 Bolton Surgical Ltd.	This document is approved for use:			Sig. 		Date: 01/06/2022									